



NON-STICK **red**

*The new bipolar forceps from KLS Martin –
with NON-STICK effect and a revolutionary
ergonomic design*

revolutionary
ergonomic
design





NON-STICK **red**

The new bipolar forceps

from KLS Martin – with NON-STICK effect
and a revolutionary ergonomic design



“With the new KLS Martin bipolar NON-STICK **red** forceps, tissue sticking to the tips of the instrument is definitely a problem of the past.

Especially for operations involving extremely fine dissection (such as parotid and thyroid interventions or microvascular tissue transfer), the use of this new generation of bipolar forceps provides significant advantages due to their exceptional haptic quality: minimized risk of procedures, gentler operating technique and shorter operating times. The tips of the instrument need to be cleaned considerably less often during interventions and the cleaning process itself is much faster as well.

We simply wouldn't like to do without these forceps.”

Dr. Paul-Stefan Mauz (M.D.)
Senior Consultant
Tübingen University Hospital
Department of Otorhinolaryngology

revolutionary
ergonomic
design

NON-STICK **red**


The new generation of bipolar forceps
with NON-STICK effect
and a revolutionary ergonomic design

The new generation of bipolar forceps

Who hasn't encountered this everyday operating situation when working with conventional bipolar forceps? The tissue tends to stick to the tips of the instrument during coagulation and when withdrawing the instrument, the coagulated tissue is torn open again.


The result is renewed bleeding. Now, this unwelcome effect can be prevented by using KLS Martin NON-STICK **red**, the new and innovative generation of bipolar forceps – the coagulated tissue is no longer disrupted when opening the instrument.

What's more, the cumbersome and time-consuming process of cleaning the tips of the forceps during operations is largely eliminated at the same time. This means fewer interruptions of the surgical procedure and corresponding time savings.

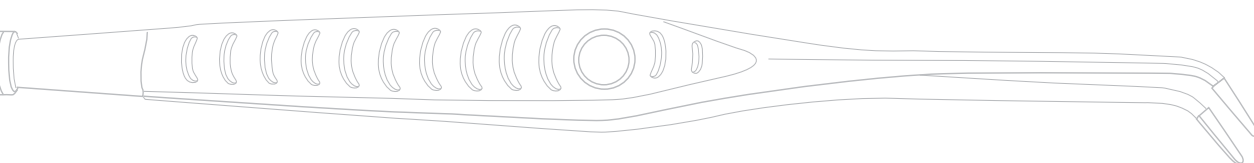

NON-STICK **red**

Standard forceps

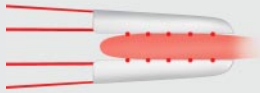


Coagulating produces so-called "hot spots".




Thanks to the special alloy used for the tips of the forceps, the heat can be discharged quickly and effectively.




This prevents the tissue from sticking to the tips of the forceps.

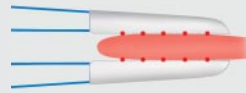

Coagulating produces so-called "hot spots".

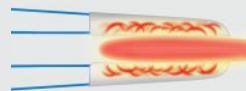


As the heat cannot be discharged, overheating is the inevitable result.

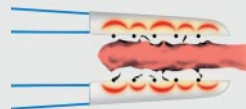

Tissue tends to stick to the tips of the forceps. When opening the instrument, charred tissue is torn off in the process.

Improved heat conductivity

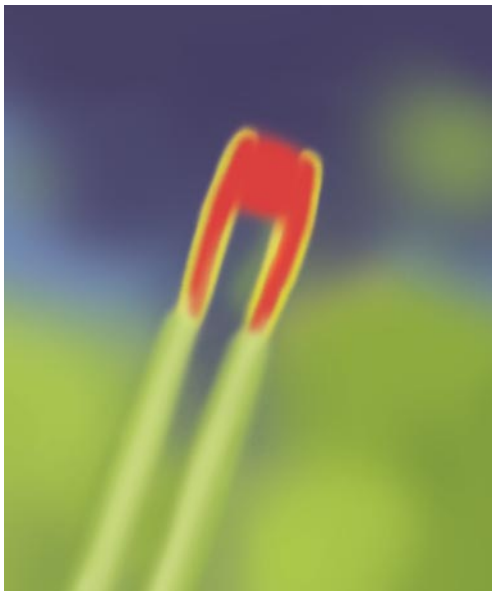


A special feature does the trick – polished noble-metal tips offering excellent heat conductivity. Thanks to this property, the tissue cannot adhere to the tips of the forceps.

The heat is discharged from the tips quickly and effectively. And what's more, the NON-STICK effect will last because the tips are not merely coated, but made of solid noble metal.

NON-STICK **red**

Variety of models



The new bipolar NON-STICK **red** forceps from KLS Martin are available in all usual sizes for the most diverse fields of use.

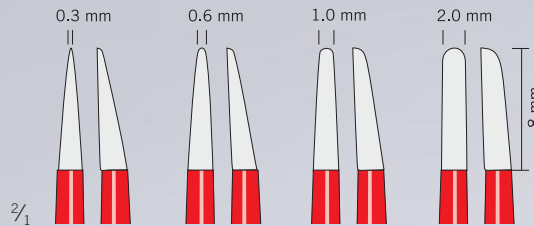

The tips of the forceps

The tips are precision-made and available in different sizes for extremely accurate surgical use. By adjusting the pressure applied, the user can coagulate merely with the tips or by using the full surfaces.

The polished noble-metal tips are excellent heat conductors preventing tissue adhesion.

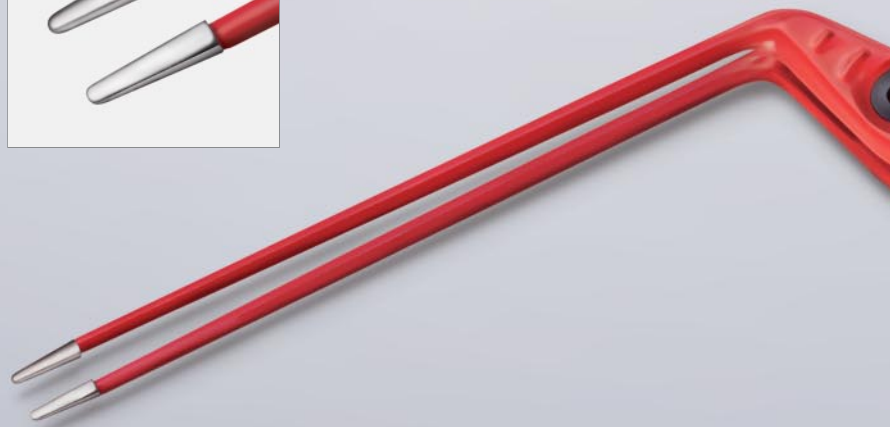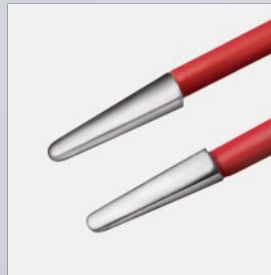


marSIGHT

The special geometry of the forceps tips gives the surgeon an unobstructed view of the surgical site and the tissue to be coagulated. Moreover, the tips enable you to grasp tissue with utmost precision. Don't miss out on the experience!

The models featuring tips with marSIGHT geometry are clearly marked in this brochure.



revolutionary ergonomic design

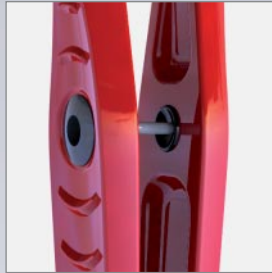

Guiding mechanism

The guiding mechanism ensures the accurate, highly parallel closure of the pre-tensioned forceps and prevents “tip spread” as well. This, in turn, prevents fatigue in the user’s hand and facilitates the dissecting process.




Connection

The bipolar NON-STICK **red** forceps can be used with any commonly available HF generator. Special connecting cables are available for this purpose.



marGRiP

The ergonomic handle surface provides for secure handling and prevents fatigue in the user’s hand.

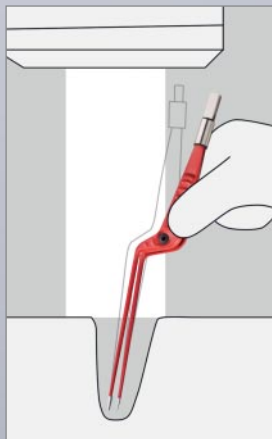

Shank geometry

Thanks to the view-optimized geometry of the shanks of the forceps, the surgeon’s hand and the cable connector are kept out of the surgical field. This is particularly advantageous when working under the operating microscope.

revolutionary
ergonomic
design

Bipolar forceps, KLS Martin NON-STICK red, straight



1/1

80-982-12-04
12 cm/4 3/4"
pointed, 0.3 mm

Example illustration


	Tip width	Length	Item No.
marSIGHT tip			
0.3 mm			
		12 cm/4 3/4"	80-982-12-04
		17 cm/6 1/4"	80-982-17-04
		20 cm/8"	80-982-20-04
		23 cm/9"	80-982-23-04
Standard tip			
0.6 mm			
		12 cm/4 3/4"	80-984-12-04
		17 cm/6 1/4"	80-984-17-04
		20 cm/8"	80-984-20-04
Standard tip			
1.0 mm			
		12 cm/4 3/4"	80-986-12-04
		17 cm/6 1/4"	80-986-17-04
		20 cm/8"	80-986-20-04
		23 cm/9"	80-986-23-04
Standard tip			
2.0 mm			
		17 cm/6 1/4"	80-988-17-04
		20 cm/8"	80-988-20-04
		23 cm/9"	80-988-23-04

revolutionary ergonomic design


Bipolar forceps, KLS Martin NON-STICK *red*,
angled




Standard tip

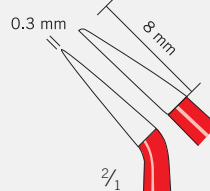


Standard tip

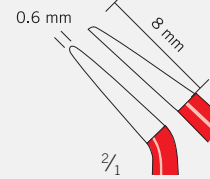

Standard tip

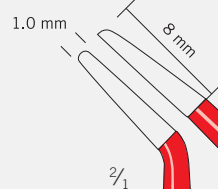

Standard tip

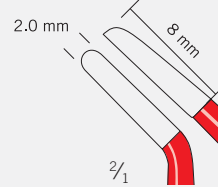

Tip width	Length	Item No.
pointed		
0.3 mm	12 cm/4 3/4"	80-983-12-04
0.3 mm	17 cm/6 1/4"	80-983-17-04
blunt		
0.6 mm	12 cm/4 3/4"	80-985-12-04
0.6 mm	17 cm/6 1/4"	80-985-17-04
0.6 mm	20 cm/8"	80-985-20-04
blunt		
1.0 mm	12 cm/4 3/4"	80-987-12-04
1.0 mm	17 cm/6 1/4"	80-987-17-04
1.0 mm	20 cm/8"	80-987-20-04
1.0 mm	23 cm/9"	80-987-23-04
blunt		
2.0 mm	17 cm/6 1/4"	80-989-17-04
2.0 mm	20 cm/8"	80-989-20-04
2.0 mm	23 cm/9"	80-989-23-04
2.0 mm	25 cm/9 3/4"	80-989-25-04
2.0 mm	28 cm/11"	80-989-28-04

1/1

80-983-12-04
12 cm/4 3/4"
pointed, 0.3 mm

Example illustration



Example illustration


revolutionary
ergonomic
design

Bipolar forceps, KLS Martin NON-STICK red, bayonet-shaped

	Tip width	Length	Item No.
marSIGHT tip			
0.3 mm			
		17 cm/6 1/4"	80-990-17-04
		20 cm/8"	80-990-20-04
		23 cm/9"	80-990-23-04
		25 cm/9 3/4"	80-990-25-04
pointed			
0.3 mm			
0.3 mm			
0.3 mm			
0.3 mm			
blunt			
0.6 mm			
0.6 mm			
0.6 mm			
0.6 mm			
Standard tip			
1.0 mm			
		17 cm/6 1/4"	80-992-17-04
		20 cm/8"	80-992-20-04
		23 cm/9"	80-992-23-04
		25 cm/9 3/4"	80-992-25-04
blunt			
1.0 mm			
1.0 mm			
1.0 mm			
1.0 mm			
Standard tip			
2.0 mm			
		17 cm/6 1/4"	80-993-17-04
		20 cm/8"	80-993-20-04
		23 cm/9"	80-993-23-04
		25 cm/9 3/4"	80-993-25-04
blunt			
2.0 mm			
2.0 mm			
2.0 mm			
2.0 mm			







Example illustration

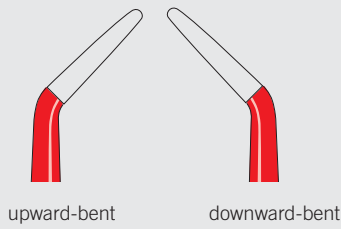

revolutionary
ergonomic
design

*Bipolar forceps, KLS Martin NON-STICK red,
bayonet-shaped, angled*

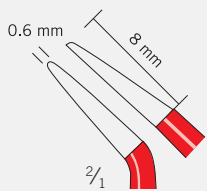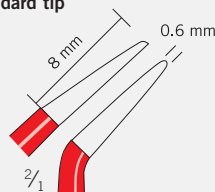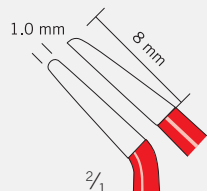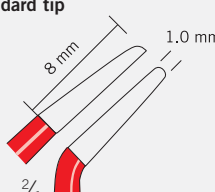	Tip width	Length	Item No.
Standard tip			
	0.6 mm	8 mm	
			$\frac{2}{1}$
blunt			
	0.6 mm	23 cm/9"	80-997-23-04
	0.6 mm	25 cm/9 $\frac{3}{4}$ "	80-997-25-04
Standard tip			
	0.6 mm	8 mm	
			$\frac{2}{1}$
blunt			
	0.6 mm	23 cm/9"	80-996-23-04
	0.6 mm	25 cm/9 $\frac{3}{4}$ "	80-996-25-04
Standard tip			
	1.0 mm	8 mm	
			$\frac{2}{1}$
blunt			
	1.0 mm	20 cm/8"	80-994-20-04
	1.0 mm	23 cm/9"	80-994-23-04
	1.0 mm	25 cm/9 $\frac{3}{4}$ "	80-994-25-04
Standard tip			
	1.0 mm	8 mm	
			$\frac{2}{1}$
blunt			
	1.0 mm	20 cm/8"	80-995-20-04
	1.0 mm	23 cm/9"	80-995-23-04
	1.0 mm	25 cm/9 $\frac{3}{4}$ "	80-995-25-04

Bipolar forceps,
KLS Martin NON-STICK *red*,
angled



*Connecting cables for bipolar forceps
with angled plugs*



80-291-40-04

4 m/13 ft.

Connecting cable for bipolar instruments
for KLS Martin and Berchtold HF units



80-286-40-04

4 m/13 ft.

Connecting cable for bipolar instruments
for maxium® "e"-version / Erbe HF units ICC
and ACC/VIO



80-293-40-04

4 m/13 ft.

Connecting cable for bipolar instruments
for maxium® "i"-version, ME MB2 "i"-version /
Valleylab HF units




Storage and transport

To store and transport your NON-STICK **red** forceps safely and securely, we recommend using the new KLS Martin mesh tray system.

A lid is optionally available for each tray size, to be easily locked in place by simply folding down the tray handles.

The forceps can be fixed in place in the tray fully in accordance with your needs.

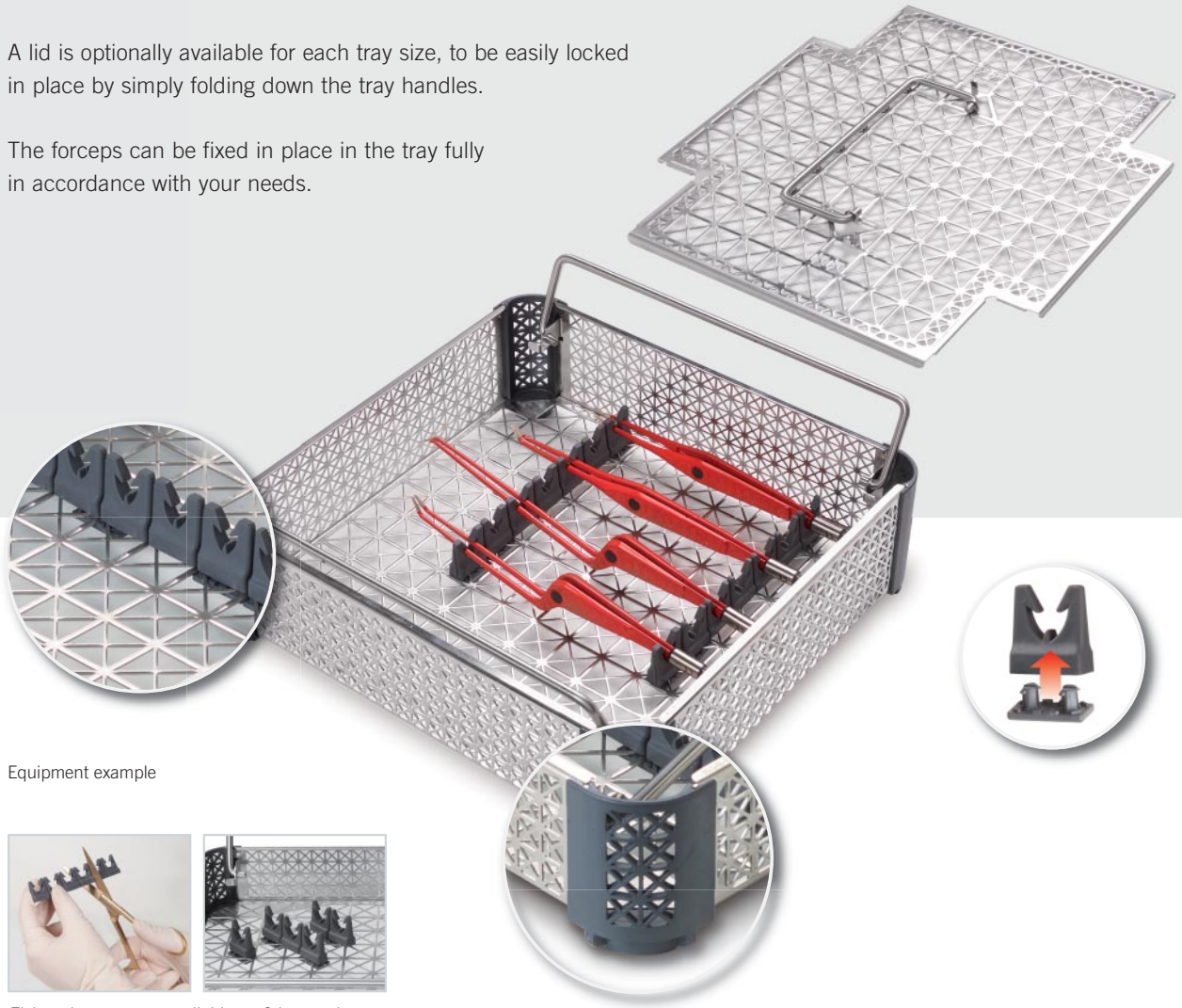

Equipment example



Fixing elements are available as 6-item strips with three holding clips. However, you can easily cut up the strips with a pair of scissors – just as you need it.

For forceps with a maximum length of 23 cm, we recommend the following configuration:

Item No.	Designation	Quantity
55-805-25-01	Mesh tray, 243 x 255 x 53 mm	1
55-805-28-01	Lid, 243 x 255 mm	1
	Fixing elements	
55-806-02-04	Clip for fixing element (10/pack)	1
55-806-07-04	Fixing element (6/pack)	2

For forceps longer than 23 cm, a ¾-size tray should be used.

Item No.	Designation	Quantity
55-805-42-01	Mesh tray, 410 x 255 x 53 mm	1
55-805-45-01	Lid, 410 x 255 mm	1
	Fixing elements	
55-806-02-04	Clip for fixing element (10/pack)	1
55-806-07-04	Fixing element (6/pack)	2



For an overview of the whole range of KLS Martin HF products, please refer to our Catalog "Accessories for electrosurgery".

Printed version
90-302-48-07



The respectful treatment of people, society and the environment has always been a matter of course to us. By design, most of our products can be recycled and used over many years, thus saving resources and reducing waste. Environment-friendly and recyclable materials are used also for our production processes, and energy and water consumption monitoring is a top priority as well. A case in point: our largest manufacturing site includes a heat recovery plant for efficient energy utilization. These and many other measures are clear evidence of our environmental commitment.

For further information, please refer to separate GoGreen brochure.

KLS Martin Group

Karl Leibinger Medizintechnik GmbH & Co. KG
78570 Mühlheim · Germany
Tel. +49 7463 838-0
info@klsmartin.com

KLS Martin GmbH + Co. KG
79224 Umkirch · Germany
Tel. +49 7461 9802-0
info@klsmartin.com

Stuckenbrock Medizintechnik GmbH
78532 Tuttlingen · Germany
Tel. +49 7461 165880
verwaltung@stuckenbrock.de

Rudolf Buck GmbH
78570 Mühlheim · Germany
Tel. +49 7463 99516-30
info@klsmartin.com

KLS Martin France SARL
68200 Mulhouse · France
Tel. +33 3 8951 3150
france@klsmartin.com

Martin Italia S.r.l.
20871 Vimercate (MB) · Italy
Tel. +39 039 605 6731
italia@klsmartin.com

Martin Nederland/Marned B.V.
1271 AG Huizen · The Netherlands
Tel. +31 35 523 4538
nederland@klsmartin.com

KLS Martin UK Ltd.
Reading RG1 3EU · United Kingdom
Tel. +44 1189 000 570
uk@klsmartin.com

Nippon Martin K.K.
Osaka 541-0046 · Japan
Tel. +81 6 62 28 9075
nippon@klsmartin.com

KLS Martin L.P.
Jacksonville, FL 32246 · USA
Tel. +1 904 641 7746
usa@klsmartin.com

Gebrüder Martin GmbH & Co. KG
Representative Office
121471 Moscow · Russia
Tel. +7 499 792-76-19
russia@klsmartin.com

Gebrüder Martin GmbH & Co. KG
Representative Office
201203 Shanghai · China
Tel. +86 21 2898 6611
china@klsmartin.com

Gebrüder Martin GmbH & Co. KG
Representative Office
Dubai · United Arab Emirates
Tel. +971 4 454 16 55
middleeast@klsmartin.com

1923
2013
YEARS | **SURGICAL**
INNOVATION

Gebrüder Martin GmbH & Co. KG
A company of the KLS Martin Group
KLS Martin Platz 1 · 78532 Tuttlingen · Germany
Postfach 60 · 78501 Tuttlingen · Germany
Tel. +49 7461 706-0 · Fax +49 7461 706-193
info@klsmartin.com · www.klsmartin.com

